

COVID-19 Rapid Testing Self-Test Kit Assembly Guide

Supplies Needed

Supplies	Provided by
Plastic bags	Organization
Positive information card (who to call, to self-isolate, and book PCR test)	Organization
Instruction pamphlets (PDF Link supplied)	PDF Link provided by DHW
Rapid test devices (analysis cassette)	Organization to print
Nasal swabs	DHW
Extraction tubes	DHW
Extraction tube caps (blue cap)	DHW
Buffer solution	DHW

Assembly Process

1. To a plastic bag, add one (1) each of the following:
 - a. rapid test device (sealed in original packaging)
 - b. nasal swab (sealed in original packaging)
 - c. instruction pamphlet
 - d. Positive information card
2. Add buffer solution to an extraction tube until the meniscus is just above the fill line (see instructions in box for more detail).
3. Seal the tube with a blue cap and firmly tighten the top and bottom caps.
4. Verify the fill level of each tube and check both caps.
5. Add the pre-filled extraction tube to the bag and seal the top of the bag.

